

IFU Size : 161 H x 104 W

ZITEMP

ZOE CEMENT QUICK SET



Product Description : Zitemp is quick set easy mix zinc oxide-eugenol composition for thermal insulation and temporary restorations.

Composition : Powder is Zinc Oxide and liquid contains Eugenol and Accelerators.

Intended Purpose / Intended Use : It is Intended to be used as a temporary filling material.

Indications for Use : Temporary restoration, temporary cementation of crowns, cavity liner under most restorative materials.

Instructions for Use : Mix Powder and liquid in a ratio of 0.32 to 0.08 (1scoop powder and 3-4 drop liquid) for about 45- 60 seconds to obtains putty consistency. Place the putty into the cavity and remove excess of the product immediately. Allow hardening for 11-12 minutes. Trim the restoration with a tungsten carbide bur. The dispensed material shall not be placed back in container and the dose once applied and used shall not be reused.

Warning : Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take internally. Contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs, immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs, discontinue the use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reaction to any of the components.

Precautions : Zitemp is a Eugenol based composition and therefore it should not be used as a liner under resin based filling materials. Avoid contact with eyes. Do not invert the bottle upside down while opening to avoid excess dispensing of the powder and avoid excessive force while dispensing the liquid from the dropper.

Contraindications : It is contraindicated for use with patient who have a history of severe allergic reaction to any of the components.

Storage Conditions : Store at temperature between 10°C(50°F) to 24°C(75°F). Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

Shelf Life : 3 years from the date of manufacturing.

Presentation:

Ref : 30016

15g Powder, 10ml Liquid, Spoon, Mixing Pad & Dispensing Dropper.

Disposal conditions : To be disposed off based on the prevailing rules of the country in which it is used.

Prevest DenPro Limited
Unit II, Export Promotion Industrial Park (EPIP),
Bari Brahmna, Jammu-181133, India.
www.prevestdenpro.com
Customer Care No. : + 91 8899074151
Email : customercare@prevestdenpro.com
Made in India

EC REP
Obelis s.a
Boulevard General Wahlen 53
1030 Brussels, Belgium
Tel: +32 2 732 99 54, Fax: +32 2 732 60 63
E-mail : mail@obelis.net

CE
0123

Revision : 10
Date : 18-11-2024

UDI Symbol for "UNIQUE DEVICE IDENTIFIER"

LOT Symbol for "BATCH CODE" USE-BY DATE Symbol for "USE-BY DATE" NON STERILE Symbol for "NON STERILE" KEEP DRY Symbol for "KEEP DRY"
CAUTION Symbol for "CAUTION" CONSULT INSTRUCTION FOR USE Symbol for "CONSULT INSTRUCTION FOR USE" REF Symbol for "CATALOGUE NUMBER"
EC REP Symbol for "AUTHORIZED REPRESENTATIVE IN THE EUROPEAN COMMUNITY/EUROPEAN UNION"
DO NOT USE IF PACKAGE IS DAMAGED Symbol for "DO NOT USE IF PACKAGE IS DAMAGED" KEEP OUT OF THE REACH OF CHILDREN Symbol for "KEEP OUT OF THE REACH OF CHILDREN" MANUFACTURER Symbol for "MANUFACTURER"
TEMPERATURE LIMIT Symbol for "TEMPERATURE LIMIT" KEEP AWAY FROM SUNLIGHT Symbol for "KEEP AWAY FROM SUNLIGHT" MD Symbol for "MEDICAL DEVICE"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding. For feedback, complaints and reporting of unexpected events contact customer care.