

Actino Gel/Actino®

Dental Etching Gel/Liquid

Actino Dental Etching Gel/Liquid is a Blue tinted viscous gel/Liquid of 37% Phosphoric acid for etching tooth enamel and dentine or composite surfaces prior to the bonding of any composites, sealants or bonding adhesives.



Indications

- Used for conditioning both dental enamel and dentine surfaces prior to the placement of an adhesive restoration.

Benefits

- 37% phosphoric acid to give best dentine etching results.
- Provides selective and controlled etching.
- Does not dry etched tooth surface.
- Gel form allows for more convenient usage and controlled application on the tooth surface.

Chemistry

Phosphoric Acid, Fillers, Solvent, Purified Water and Coloring Agent.


Technical Data

- 1) Colour : Blue
- 2) PH : <1
- 3) Phosphoric Acid : 37%

Techniques



Cavity Preparation

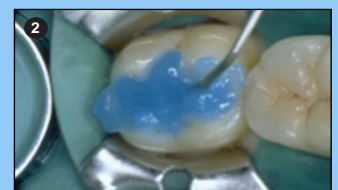

Placement of Actino Gel and allow the gel to stay for 10 to 15 second.



Rinse thoroughly with water spray



Dry etched tooth surface with clean, dry air / cotton swab.

Presentation

- Ref : 10006-1 (**Trial Pack** : 1 x 2ml Syringe, 5 x Applicator Tips, 1 x Instruction Manual)
 Ref : 10006-2 (**Trial Pack** : 1 x 3ml Syringe, 10 Applicator Tips, 1 x Instruction Manual)
 Ref : 10006 (**Intro Pack** : 1 x 5ml Syringe, 10 Applicator Tips, 1 x Instruction Manual)
 Ref : 10007 (**Economy Pack** : 4 x 5ml Syringe, 40 Applicator Tips, 1 x Instruction Manual)
 Ref : 10008 (**Jumbo Pack** : 1 x 50 ml Syringe Actino Gel, 2 x Empty Syringes 5 ml, 1 x 20 Applicator Tips, 1 x Instruction Manual)
 Ref : 10009 (**Liquid Packing** : 10ml Plastic dropper bottle, 1 x Instruction Manual)

PREVEST DENPRO LIMITED

Unit II, Export Promotion Industrial Park (EPIP),
Bari Brahmana, Jammu-181133, India.

Tel.: + 91 191 350 6858, 60, 61
Tel.: + 91 8899074151

E-mail : info@prevestdenpro.com
Website : www.prevestdenpro.com

Actino Gel/Actino

Dental Etching Gel/Liquid



Actino Dental Etching Gel/Liquid is a Blue tinted viscous gel/Liquid of 37% Phosphoric acid for etching tooth enamel and dentine or composite surfaces prior to the bonding of any composites, sealants or bonding adhesives.

PREVEST DENPRO LIMITED

Unit II, Export Promotion Industrial Park (EPIP),
Bari Brahmana, Jammu-181133, India.

Tel.: + 91 191 350 6858, 60, 61
Tel.: + 91 8899074151

E-mail : info@prevestdenpro.com
Website : www.prevestdenpro.com