

Micron Bioactive

Radiopaque Hydroxyapatite
Modified-Glass Ionomer Cement, Restorative

Micron Bioactive is a radiopaque, hydroxyapatite modified-GLASS IONOMER CEMENT for restoration of primary teeth, core build up, restoration of class I, limited class II cavities and as a base and liner material. Addition of hydroxyapatite considerably improves the mechanical properties, as well as the chemical bonding, ensuring a tight seal at the dentinal margin. Micron Bioactive is radiopaque for easy post operative diagnosis.



Indications

- Core build-up.
- Restoration of primary teeth.
- Cases where radiopacity is required.
- As a base or liner.
- Class I, limited Class II, temporary fillings, etc.
- Long term temporary replacement of cusps.
- Intermediate restoratives and base material for class I and II cavities using the sandwich technique.
- Class V restorations in non-aesthetic situations.
- Restorative in the field using the ART technique.
- Temporary endodontic fillings.

Benefits

- Anticariogenic.
- Excellent chemical bonding to dentin and enamel.
- Tooth coloured.
- Superior strength and fully biocompatible.
- Accelerated remineralisation.
- Radiopaque for easy post operative diagnosis.

Chemistry

Powder : Fluoro Alumino Silicate Glass Powder, Hydroxyapatite Powder.

Liquid : Polyacrylic acid Solution.

Technical Data

- 1) Mixing Ratio : 2 : 3
- 2) Setting Time : 1:30 – 2:00 (min : sec)
- 3) Compressive Strength : 150 MPa

Techniques



1
Tooth preparation wash and dry but DO NOT DESICCATE.



2
Use two scoop of powder and three drop of liquid on a clean mixing pad. Incorporate the powder into the liquid component in 2 increments.



3
Freshly mixed cement.



4
Restoration of the prepared cavity.

Presentation

Ref. : 30009-1
1 x 15g Powder, 1 x 10ml Liquid,
1 x Mixing Pad, 1 x Plastic Spoon.
1 x Instruction Manual

PREVEST DENPRO LIMITED

Unit II, Export Promotion Industrial Park (EPIP),
Bari Brahmana, Jammu-181133, India.

Tel.: + 91 191 350 6858, 60, 61
Tel.: + 91 8899074151

E-mail : info@prevestdenpro.com
Website : www.prevestdenpro.com

Micron Bioactive

Radiopaque Hydroxyapatite
Modified-Glass Ionomer Cement, Restorative



Micron Bioactive is a radiopaque, hydroxyapatite modified-GLASS IONOMER CEMENT for restoration of primary teeth, core build up, restoration of class I, limited class II cavities and as a base and liner material. Addition of hydroxyapatite considerably improves the mechanical properties, as well as the chemical bonding, ensuring a tight seal at the dentinal margin. Micron Bioactive is radiopaque for easy post operative diagnosis.

PREVEST DENPRO LIMITED

Unit II, Export Promotion Industrial Park (EPIP),
Bari Brahmana, Jammu-181133, India.

Tel.: + 91 191 350 6858, 60, 61
Tel.: + 91 8899074151

E-mail : info@prevestdenpro.com
Website : www.prevestdenpro.com