

Orafil® LC

Light cured, temporary filling material

Orafil LC is light curing, ready to use temporary filling material which remains elastic after curing and can be easily removed in one piece with a suitable instrument without causing any damage to the prepared cavity. It is also suitable for long term temporaries.

After light curing it absorbs moisture and slightly swells to provide excellent sealing of margins. Due to one piece removal no debris is left on the cavity floor and ensure health of the gingival tissue.

With its non eugenol formula Orafil LC is suitable for inlays, onlays, endodontic access temporaries and implant screw access temporaries.



Indications

- All kinds of temporary fillings, temporary protection of inlays / onlays preparations.
- Implant screw access temporaries.
- Emergency repair of lost fillings and fractured teeth.

Benefits

- Quick and easy removal-can be removed in one piece-thus leave cavity clean upon removal.
- High wear resistance.
- Excellent adhesion
- Durable and tight marginal seal.
- Suitable for long term temporaries.
- Non eugenol composition.


Chemistry

Urethane Dimethylacrylate, Fillers, Photoinitiators, Stabilizers, Accelerators and Pigment.


Technical Data

- 1) Appearance : Dark blue homogenous paste
- 2) Depth of cure : 4-5 mm
- 3) Curing Time : 20-40 sec.


Techniques

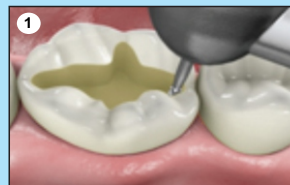


1 Tooth preparation wash and dry.

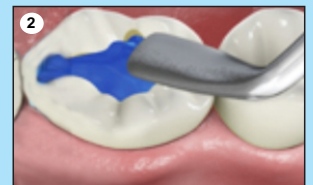

2 Fill cavity with Orafil LC.

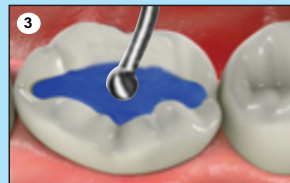


3 Condense with suitable instrument.

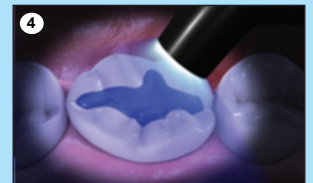

4 Light cure for 20 sec.



5 6-7 days later.

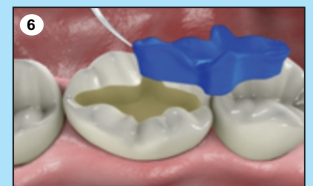

6 Remove filling with suitable instrument.

Presentation

Ref. : 11002-1
1 x 3g Syringe, 1 x Instruction Manual

PREVEST DENPRO LIMITED

Unit II, Export Promotion Industrial Park (EPIP),
Bari Brahmana, Jammu-181133, India.

Tel.: + 91 191 350 6858, 60, 61
Tel.: + 91 8899074151

E-mail : info@prevestdenpro.com
Website : www.prevestdenpro.com

Orafil® LC

Light cured, temporary filling material



Orafil LC is light curing, ready to use temporary filling material which remains elastic after curing and can be easily removed in one piece with a suitable instrument without causing any damage to the prepared cavity. It is also suitable for long term temporaries.

After light curing it absorbs moisture and slightly swells to provide excellent sealing of margins. Due to one piece removal no debris is left on the cavity floor and ensure health of the gingival tissue.

With its non eugenol formula Orafil LC is suitable for inlays, onlays, endodontic access temporaries and implant screw access temporaries.

PREVEST DENPRO LIMITED

Unit II, Export Promotion Industrial Park (EPIP),
Bari Brahmana, Jammu-181133, India.

Tel.: + 91 191 350 6858, 60, 61
Tel.: + 91 8899074151

E-mail : info@prevestdenpro.com
Website : www.prevestdenpro.com