

OraTemp C&B

Temporary Crown and Bridge Material

OraTemp C&B (crown and bridge material) is a two component self curing composite material for chair side fabrication of temporary crowns, bridges and other temporary intra-coronal restorations. Ideal for both short and long term temporization.



Indications

- Fabrication of temporary crowns, bridges, inlays, onlays, and veneers.
- Fabrication of long-lasting temporary restorations.
- Lining material for prefabricated temporary crowns made of composite.

Benefits

- Excellent strength and low wear for durable temporaries.
- Superior surface details.
- Extremely polishable and colour stable.
- 1 : 1 volume ratio for an even flow and improved handling properties.
- Low shrinkage and low curing temperature.
- Can be easily repaired with flowable composite.

Chemistry

Consists of Methacrylate monomers and fillers.


Presentation

Ref. : 11005
1 x 67g Cartridge (Base + Catalyst Paste), 16 x Applicator Tips,
1 x Instruction Manual.

Technical Data

- 1) Working Time : 1 min - 1 min 20 sec.
- 2) Compressive Strength : 300 MPa
- 3) Setting Time : 2 min - 2 min 30 sec.
- 4) Flexural Strength : 100 MPa


Techniques

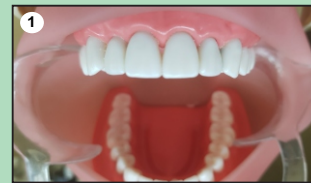


Pre preparation model.




Putty index in place.

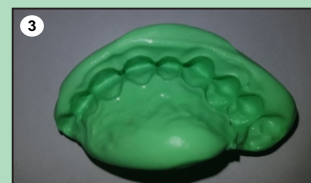


Tissue surface of putty index.

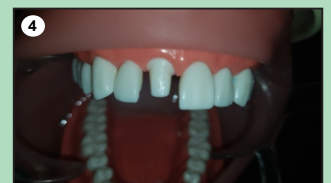

Prepared tooth on model.

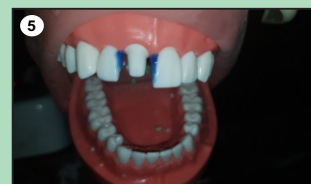

Block out of adjacent undercuts with Ultra Bloc VLC.

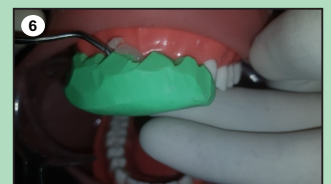

Putty index loaded with OraTemp C&B.



Provisional crown just after removal of index.



Provisional crown after finishing and polishing.

PREVEST DENPRO LIMITED

Unit II, Export Promotion Industrial Park (EPIP),
Bari Brahmana, Jammu-181133, India.

Tel.: + 91 191 350 6858, 60, 61
Tel.: + 91 8899074151

E-mail : info@prevestdenpro.com
Website : www.prevestdenpro.com

OraTemp C&B

Temporary Crown and Bridge Material



OraTemp C&B (crown and bridge material) is a two component self curing composite material for chair side fabrication of temporary crowns, bridges and other temporary intra-coronal restorations. Ideal for both short and long term temporization.

PREVEST DENPRO LIMITED

Unit II, Export Promotion Industrial Park (EPIP),
Bari Brahmna, Jammu-181133, India.

Tel.: + 91 191 350 6858, 60, 61
Tel.: + 91 8899074151

E-mail : info@prevestdenpro.com
Website : www.prevestdenpro.com