

Renew Universal®

One Component Self Etch Universal Bonding Agents

Renew Universal is a light cured single component dental bonding agent containing-etching, priming and bonding in one bottle and contains unique combination such as 10-MDP and 4-META. Compatible with light cured and dual cured composite materials.



Indications

- Direct bonding of light-cured composites, compomers and ormocers to tooth structure.
- Bonding of dual-cured core build up composites to tooth structure as long as these materials are light-cured.
- Intraoral repair of composite, metal-based and zirconia/alumina-based restorations.
- Treatment of hypersensitive teeth.

Benefits

- Excellent bonding capabilities to dentin and cut /uncut enamel.
- Fully compatible with light cured and dual cured composite materials.


Chemistry

A novel dental adhesive containing a unique combination of functional monomers such as 10-MDP And 4-META.


Technical Data

- 1) Curing Time : 20-30sec
- 2) Shear Bond Strength : 15-20 Mpa


Techniques

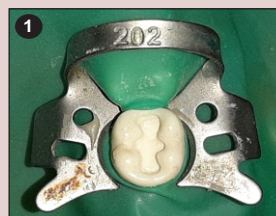

Prepared cavity

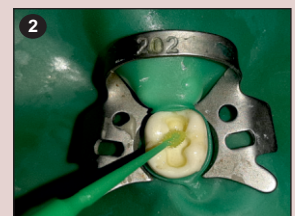

Apply & leave undisturbed for 10 sec after the application

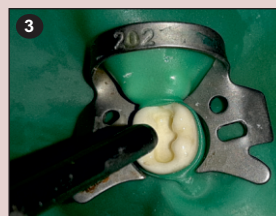

Dry thoroughly for 5 sec with oil free air



Light cure for 20 sec

Presentation

- Ref. : 10017 **Intro Pack** : 1 x 1.2ml Syringe, 5 x Applicator Tips, 1 x Instruction manual.
Ref. : 10018 **Regular Pack** : 1 x 5ml Bottle, 1 x Instruction manual.
Ref. : 10019 **Kit Pack** : 2 x 5ml Bottle, 40 x Micro Brushes, 1 x Instruction manual.

PREVEST DENPRO LIMITED

Unit II, Export Promotion Industrial Park (EPIP),
Bari Brahmana, Jammu-181133, India.

Tel.: + 91 191 350 6858, 60, 61
Tel.: + 91 8899074151

E-mail : info@prevestdenpro.com
Website : www.prevestdenpro.com

Renew Universal®

One Component Self Etch Universal Bonding Agents



Renew Universal is a light cured single component dental bonding agent containing-etching, priming and bonding in one bottle and contains unique combination such as 10-MDP and 4-META. Compatible with light cured and dual cured composite materials.

PREVEST DENPRO LIMITED

Unit II, Export Promotion Industrial Park (EPIP),
Bari Brahmana, Jammu-181133, India.

Tel.: + 91 191 350 6858, 60, 61
Tel.: + 91 8899074151

E-mail : info@prevestdenpro.com
Website : www.prevestdenpro.com