

ViroWipes

PREVESTDenPro®
THE FUTURE OF DENTISTRY

Rapid Disinfection Wipes for Medical Devices

Product Description : ViroWipes is a one-step, ready to use disposable wipe for the rapid disinfection of all kinds of wipeable surfaces of non-invasive medical devices such as hand pieces, contra-angle, turbines, scaler inserts and other non invasive instruments and equipments with increased risk of infection.

Composition :

N-(3-aminopropyl)-N-dodecylpropane-1, 3-Diamine, Quaternary Ammonium Compound, Surfactant , IPA, pH modifier, Purified water

Intended Purpose/Intended Use: It is intended to be used for cleaning, decontaminating and disinfecting non invasive, reusable medical devices and surfaces.

Indications for Use:

It is used for a rapid disinfection and cleaning of medical devices and surfaces like hand pieces, contra-angles, turbines, scaler inserts etc.

Instructions for Use : Wipe the equipment with ViroWipes and massage thoroughly over the surface in scrubbing action to ensure good surface contact with the disinfectant. Leave the instruments/equipments to dry. Do not rinse with water and do not mix with any other disinfectant.

Warnings :

IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water/ shower.

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses if present and continue rinsing. Get medical advice/attention if you feel unwell.

Precautions :

- ♦ Advised for use on surfaces of medical devices by professionals only.
- ♦ Wear personal protective equipment such as gloves/protective clothing/eye protection/ masks for face protection.
- ♦ Wash your hands after handling.
- ♦ Do not use in combination with other products.
- ♦ Single use, not reusable.
- ♦ The product is not suitable for use on alcohol sensitive devices. Before application, check the compatibility of surfaces with alcohol base products.

Contraindications : Avoid heat

Storage Conditions : Store at temperature between 10°C(50°F) to 24°C(75°F). Close the cap carefully after use. The shelf life of the product is valid only if left unopened and stored in a cool and dry place.

Shelf life : 3 years from the date of manufacturing.

Presentation :

Ref : 16015

50 Wipes (10x15) in a Tub

Ref : 16016

50 Wipes (15x20) in a Tub

Ref : 16017

80 Wipes (15x20) in a Tub

Disposal Conditions : To be disposed off based on the prevailing rules of the country in which it is used.

Prevest DenPro Limited

Unit II, Export Promotion Industrial Park

(EPIP), Bari Brahmna, Jammu-181133,

India. www.prevestdenpro.com

Customer Care No. : + 91 8899074151

Email : customercare@prevestdenpro.com




Made in India

 Symbol for "UNIQUE DEVICE IDENTIFIER"

 Symbol for "BATCH CODE"  Symbol for "USE-BY DATE"  Symbol for "NON STERILE"  Symbol for "KEEP DRY"

 Symbol for "CAUTION"  Symbol for "CONSULT INSTRUCTION FOR USE"  Symbol for "CATALOGUE NUMBER"

 Symbol for "AUTHORIZED REPRESENTATIVE IN THE EUROPEAN COMMUNITY/EUROPEAN UNION"

 Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"  Symbol for "KEEP OUT OF THE REACH OF CHILDREN"  Symbol for "MANUFACTURER"

 Symbol for "TEMPERATURE LIMIT"  Symbol for "KEEP AWAY FROM SUNLIGHT"  Symbol for "MEDICAL DEVICE"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding. For feedback, complaints and reporting of unexpected events contact customer care.