

# ORADOX

## Effervescent Mouthwash Tablet

**PREVESTDenPro®**  
THE FUTURE OF DENTISTRY

**Product Description :** Oradox, an antiseptic effervescent mouthwash tablet works within 1 minute and facilitates deep cleansing of oral cavity before and after dental treatment. It is fast, simple and effective. The unique combination of ingredients with safest surfactants removes impurities, thick saliva and bloods and tooth debris from the operative sites. The formula has a fresh flavor with odour-free and non-irritant feeling, giving the patient a sense of well being. The mouthwash tablet can also be used for regular use at home. The tablets are available in different color and flavor combinations.

**Composition :** Sodium Bicarbonate, Citric acid, Thymol, Flavouring agents and Preservatives.

**Intended purpose/intended use :** It is intended to be used as mouthwash tablets.

**Indication for use:** Oradox mouthwash tablets can be used after all dental procedures to facilitate the cleaning of teeth and gums. The optimized tablet formula is especially recommended for :

- ◆ Freshness and cleaning of mouth before, during and after dental treatment.
- ◆ The removal of debris and saliva from the oral cavity.
- ◆ The removal of blood and tooth debris from the operative site essentially during and after the dental treatment.

**Instructions for use :**

Drop one or two Oradox tablet(s) into a tumbler full of clean tap water (approximately 200ml). Within 1-2 minutes, tablet dissolves with effervescence making the solution sweet and flavoured. Ask the patient to take a mouthful and rinse the solution around the teeth and gums for up to 30 seconds and spit out. Repeat as required.

**Warnings :**

Do not take internally. If the product solution comes into contact with eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reaction to any of the components.

**Precautions :**

- ◆ Keep tablets out of reach of children and those at risk of accidentally swallowing the tablet or solution.
- ◆ Do not swallow the solution or tablet.

**Contraindications :**

Do not use on patients with known allergies to any of the ingredients.

**Storage Conditions :**

Keep the bottle in cool and dry place maintained at Temp. 10°C(50°F) -24°C(75°F). After use tightly screw cap the bottle and do not allow the unused tabs to come in contact with moisture.

**Shelf Life :**

2 years from the date of manufacturing.

**Presentation :**

Ref : 80011

1 x 100 Tablets Bottle (Mint Flavour)

Ref : 80008

1 x 250 Tablets Bottle (Mint Flavour)

Ref : 80009

1 x 500 Tablets Bottle (Mint Flavour)

Ref : 80010

1 x 1000 Tablets Bottle (Mint Flavour)

Ref : 80012

1 x 100 Tablets Bottle (Fresh Lime Flavour)

Ref : 80013

1 x 250 Tablets Bottle (Fresh Lime Flavour)

Ref : 80014

1 x 500 Tablets Bottle (Fresh Lime Flavour)

Ref : 80015

1 x 1000 Tablets Bottle (Fresh Lime Flavour)

Color : Green/Peppermint, Yellow/Lemon.

**Disposal conditions :**

To be disposed off based on the prevailing rules of the country in which it is used.

**Prevest DenPro Limited**  
Unit II, Export Promotion Industrial Park (EPIP),  
Bari Brahmna, Jammu-181133, India.  
[www.prevestdenpro.com](http://www.prevestdenpro.com)  
Customer Care No. : + 91 8899074151  
Email : [customercare@prevestdenpro.com](mailto:customercare@prevestdenpro.com)  
Made in India

CE

Revision :	00
Date :	12-04-2023

 Symbol for "UNIQUE DEVICE IDENTIFIER"

 Symbol for "BATCH CODE"  Symbol for "USE-BY DATE"  Symbol for "NON STERILE"  Symbol for "KEEP DRY"

 Symbol for "CAUTION"  Symbol for "CONSULT INSTRUCTION FOR USE"  Symbol for "CATALOGUE NUMBER"

 Symbol for "AUTHORIZED REPRESENTATIVE IN THE EUROPEAN COMMUNITY/EUROPEAN UNION"

 Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"  Symbol for "KEEP OUT OF THE REACH OF CHILDREN"  Symbol for "MANUFACTURER"

 Symbol for "TEMPERATURE LIMIT"  Symbol for "KEEP AWAY FROM SUNLIGHT"  Symbol for "MEDICAL DEVICE"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding. For feedback, complaint and reporting of unexpected events contact customer care.