

ORADOX REVEAL

Disclosing Tabs



Product Description : Oradox Reveal are the disclosing tablets which highlights dental plaque, thus aids in effective oral prophylaxis and improve brushing habits.

Composition : Xylitol, PVP, Citric acid, Sodium benzoate, Excipients, Coloring and Flavoring agent

Intended Purpose / Intended Use : It is intended to be used as a disclosing agent to stain plaque biofilm.

Indications for Use : These chewable tablets are used to stain dental plaque and make it visible to naked eye.

Instructions for Use :

A. Improving Brushing :

1. Brush and floss your teeth and rinse mouth with water.
2. Chew Oradox Reveal tablet, swish around for 30 seconds and spit it out.
3. Check for purple stains on the teeth as these stains signifies that plaque is still present on the teeth.
4. Brush a second time more thoroughly to clean all the remaining plaque biofilm and then rinse with water.
5. Rinse tooth brush and sink after use to help prevent stains.

B. Aid Oral Prophylaxis :

1. Chew Oradox Reveal tablet, swish around for 30 seconds and spit it out. Rinse with water.
2. Check for purple stains on the teeth as these stains signifies that plaque is still present on the teeth.
3. Perform a through oral prophylaxis to eliminate the plaque.

Warnings : Harmful if swallowed.

Causes severe nose irritation, skin burns and eye damage.

IF SWALLOWED: Rinse mouth. Do NOT induce vomiting.

IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water/shower.

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses if present and continue rinsing.

Get medical advice/attention if you feel unwell.

Keep out of reach of children.

Precautions :

Do not swallow the tablets.

Avoid stains on clothing as the coloring agent is strong.

Contraindications : Do not use on patients with known allergies to any of the ingredients.

Storage Conditions : Store at temperature between 10°C(50°F) to 24°C(75°F). Keep away from moisture and heat. Close the cap carefully after use. The shelf life of the product is valid only if left unopened and correctly stored in a cool dry place.

Shelf Life : 2 years from the date of manufacturing.

Presentation :

Ref: 80030

1 x 30 Tabs in Flip Top container (Blueberry)

Ref. : 80031

1 x 60 Tabs in Flip Top container (Blueberry)

Disposal Conditions : To be disposed off based on the prevailing rules of the country in which it is used.

Prevest DenPro Limited

Unit II, Export Promotion Industrial Park,
(EPIP), Bari Brahmana, Jammu-181133,
India. www.prevestdenpro.com

Customer Care No. : + 91 8899074151

Email : customercare@prevestdenpro.com

Made in India

CE

Revision :	00
Date :	28-07-2023

UDI Symbol for "UNIQUE DEVICE IDENTIFIER"

LOT Symbol for "BATCH CODE" USE-BY DATE Symbol for "USE-BY DATE" NON STERILE Symbol for "NON STERILE" KEEP DRY Symbol for "KEEP DRY"

CAUTION Symbol for "CAUTION" CONSULT INSTRUCTION FOR USE Symbol for "CONSULT INSTRUCTION FOR USE" REF Symbol for "CATALOGUE NUMBER"

EC REP Symbol for "AUTHORIZED REPRESENTATIVE IN THE EUROPEAN COMMUNITY/EUROPEAN UNION"

DO NOT USE IF PACKAGE IS DAMAGE Symbol for "DO NOT USE IF PACKAGE IS DAMAGE" KEEP OUT OF THE REACH OF CHILDREN Symbol for "KEEP OUT OF THE REACH OF CHILDREN" MANUFACTURER Symbol for "MANUFACTURER"

TEMPERATURE LIMIT Symbol for "TEMPERATURE LIMIT" KEEP AWAY FROM SUNLIGHT Symbol for "KEEP AWAY FROM SUNLIGHT" MD Symbol for "MEDICAL DEVICE"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding. For feedback, complaints and reporting of unexpected events contact customer care.