

CALCIUM HYDROXIDE POWDER, EXTRA FINE

PREVESTDenPro®
THE FUTURE OF DENTISTRY

Product Description : Calcium Hydroxide Powder is a multipurpose dental material, when mixed with deionized water, forms a smooth paste to be used as a dressing for apexification procedures, direct & indirect pulp capping and as a sub base under restoration.

Composition : Calcium Hydroxide Powder

Intended Purpose / Intended use : It is used in direct & indirect pulp capping, as sub base under restoration & dressing for apexification.

Indications for use:

- ♦ Direct & indirect pulp capping.
- ♦ Sub base under restoration.
- ♦ Dressing for apexification procedures.
- ♦ Thermal insulation.

Instructions for use : Mix the powder with deionized water to form homogenous paste and place into tooth cavity. The dispensed material shall not be placed back in the container and the dose once applied or used shall not be reused.

Warnings : Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take internally. Contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs, immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs, discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used in patients who have a history of severe allergic reaction to any of the components.

Precautions : Avoid contact with eyes and skin.

Contraindications : Do not use in patients with a known allergy to any of its components.

Storage Conditions :

Store at temperature between 10°C(50°F) to 24°C(75°F). Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

Shelf life : 3 years from the date of manufacturing.

Presentation :

Ref: 40029

1 x10g Plastic Jar.

Disposal conditions :

To be disposed off based on the prevailing rules of the country in which it is used.

Prevest DenPro Limited

Unit II, Export Promotion Industrial Park,
(EPIP), Bari Brahmam, Jammu-181133,
India. www.prevestdenpro.com

Customer Care No. : + 91 8899074151

Email : customer@prevestdenpro.com

Made in India



CE
0123

Issue/Revision :	01/00
Date :	26-05-2025

UDI Symbol for "UNIQUE DEVICE IDENTIFIER"

LOT Symbol for "BATCH CODE"  Symbol for "USE-BY DATE"  Symbol for "NON STERILE"  Symbol for "KEEP DRY"

 Symbol for "CAUTION"  Symbol for "CONSULT INSTRUCTION FOR USE" **REF** Symbol for "CATALOGUE NUMBER"

EC/REP Symbol for "AUTHORIZED REPRESENTATIVE IN THE EUROPEAN COMMUNITY/EUROPEAN UNION"

 Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"  Symbol for "KEEP OUT OF THE REACH OF CHILDREN"  Symbol for "MANUFACTURER"

 Symbol for "TEMPERATURE LIMIT"  Symbol for "KEEP AWAY FROM SUNLIGHT" **MD** Symbol for "MEDICAL DEVICE"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding. For feedback, complaint and reporting of unexpected events contact customer care.