

CHLOR X

2% CHLORHEXIDINE SOLUTION

PREVESTDenPro®
THE FUTURE OF DENTISTRY

Product Description :

Chlor X is a chlorhexidine digluconate based water - alcoholic solution for irrigation and cleaning of root canals.

Composition :

Chlorhexidine Digluconate, Solvent, Coloring agent and Water

Intended Purpose / Intended Use : It is used for irrigation and cleaning of root canals.

Indications for use : It is used for root canal disinfection, as a final endodontic rinse prior to canal obturation and prior to pulp-capping.

Instructions for use :

After root canal preparation, select desired size syringe and safe-ended irrigation needle. Fill syringe with Chlor X solution. Attach irrigation needle tip onto the syringe. Irrigate the canal. Discard irrigation needle tip & syringe after use. The dispensed material shall not be placed back in the container and the dose once applied or used shall not be reused.

Warnings :

1. Keep out of reach of children.
2. Avoid contact with skin and eyes.
3. Product containing chlorhexidine must not be mixed with products containing sodium hypochlorite to prevent formation of brown precipitate.

Precautions :

1. Read IFU for material handling and safety information.
2. Do not ingest or inject.
3. Not to be used as an oral rinse.

Contraindications :

Do not use in patients with known Hypersensitivity to chlorhexidine digluconate.

Storage Conditions :

Store at temperature between 10°C(50°F) to 24°C(75°F). Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place. Keep away from direct heat.

Shelf Life :

3 years from the date of manufacturing.

Presentation :

Ref : 40026

1 x 100ml Bottle, Dispensing Dropper.

Ref : 40026-1

1 x 500ml Bottle.

Disposal conditions : To be disposed off based on the prevailing rules of the country in which it is used.

Prevest DenPro Limited

Unit II, Export Promotion Industrial Park,
(EPIP), Bari Brahmana, Jammu-181133,
India. www.prevestdenpro.com

Customer Care No. : + 91 8899074151

Email : customercare@prevestdenpro.com

Made in India

Issue/Revision :	01/00
Date :	26-05-2025

UDI Symbol for "UNIQUE DEVICE IDENTIFIER"

LOT Symbol for "BATCH CODE"  Symbol for "USE-BY DATE"  Symbol for "NON STERILE"  Symbol for "KEEP DRY"

 Symbol for "CAUTION"  Symbol for "CONSULT INSTRUCTION FOR USE" **REF** Symbol for "CATALOGUE NUMBER"

CE Symbol for "AUTHORIZED REPRESENTATIVE IN THE EUROPEAN COMMUNITY/EUROPEAN UNION"

 Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"  Symbol for "KEEP OUT OF THE REACH OF CHILDREN"  Symbol for "MANUFACTURER"

 Symbol for "TEMPERATURE LIMIT"  Symbol for "KEEP AWAY FROM SUNLIGHT" **MD** Symbol for "MEDICAL DEVICE"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding. For feedback, complaints and reporting of unexpected events contact customer care.