

Accusil Bite Registration

Hydrophilic Vinyl Polysiloxane Impression Material, Bite Registration



Product description : Accusil Bite Registration is a vinylpolysiloxane (addition silicone), also known as A-Silicones especially made for bite registration work for crown and bridge, edentulous areas, implant techniques and laboratory procedures

Composition :

Base: Polydimethyl Siloxane Polymer, Fillers, Viscosity Modifiers, Inhibitors and Pigments.

Catalyst: Polydimethyl Siloxane Polymer, Fillers, Platinum Catalyst, Viscosity Modifiers.

Intended Purpose / Intended use: It is used for making occlusal registrations.

Indications for use: It is indicated for highly accurate bite registration techniques that enable to easily replicate correct intercuspals and centric relationships.

Instructions for use :

Pull off the cartridge plug and discard it. Eject a small amount of material directly out of the cartridge onto a paper towel/tissue held in your hand. Extrude enough material until it is evident that base and catalyst are being ejected. This ensures proper mixing. Immediately wipe the cartridge orifice to remove excess material. Attach the mixing tip to the cartridge by aligning the notch inside the tip with the cartridge orifice. Seat the tip onto the cartridge and twist a ¼ turn clockwise (90°) to lock it in place. Push the Oral tip if needed firmly onto the end of the mixing tip. Squeeze the trigger with a smooth, steady force to mix and dispense material. Avoid abrupt movements. Release the trigger to stop the flow. The dispensed material shall not be placed back in container and the dose once applied and used shall not be reused.

TECHNICAL DATA	
Color	Green
Classification ISO 4823	Type B
Mixing ratio (Base : Catalyst)	1:1
Working Time	40 sec
Setting Time	1:30 to 2:00 Min: Sec

Bite registration:

1. For bite registration, ensure the patient is brought into maximum intercuspation or a desired position during the initial working time of 40 seconds.
2. Once maximum intercuspation has been achieved, have the patient remain in this position for 90 seconds max to ensure a total set of the material.
3. Remove the material from the patient's mouth.
4. After use, store the cartridges with used mixing tips attached, to protect the remaining materials from contamination. The material can be disinfected immediately.

Warnings :

Chairside materials should be brought to room temperature prior to use. Keep away from sunlight/heat. Prolonged exposure to temperature above specified limit i.e., (10°C - 24°C) can damage the material.

Precautions:

1. Due to rapid polymerization, the patient may experience a physiologically harmless temperature increase.
2. Never attempt to extrude the material through a cured mixing canula.
3. Always disinfect cartridges between patients.
4. Store at room temperature.
5. Latex or Sulfur may affect the set of material.
6. May stain clothing.
7. Do not combine with condensation curing silicones, polyether or polysulfide.

Contraindications:

The use of Accusil Bite Registration is contraindicated if the patient is known to be allergic to any of its ingredients.

Storage conditions:

Store at temperature between 10°C(50°F) to 24°C(75°F). Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

Shelf Life: 3 years from the date of manufacturing

Presentation:

Ref. 70014

2 x 50ml Base/Catalyst Paste Cartridge, 10 x Mixing Tips

Ref. 70014-1

1 x 50ml Base/Catalyst Paste Cartridge, 5 x Mixing Tips

Disposal conditions:

To be disposed off based on the prevailing rules of the country in which it is used.

Prevest DenPro Limited
Unit II, Export Promotion Industrial Park,
(EPIP), Bari Brahmna, Jammu-181133,
India. www.prevestdenpro.com
Customer Care No. : + 91 8899074151
Email : customercare@prevestdenpro.com
Made in India

E	C	R	E	P
Obelis s.a				
Boulevard General Wabis 53				
1030 Brussels, Belgium				
Tel: +(32) 2 732 59 54, Fax: +(32) 2 732 60 03				
E-mail : mail@obelis.net				

Revision :	02
Date :	03-05-2025

UDI Symbol for "UNIQUE DEVICE IDENTIFIER"

LOT Symbol for "BATCH CODE" Symbol for "USE-BY DATE" Symbol for "NON-STERILE" Symbol for "KEEP DRY"

Symbol for "CAUTION" Symbol for "CONSULT INSTRUCTIONS FOR USE" Symbol for "CATALOGUE NUMBER"

Symbol for "AUTHORIZED REPRESENTATIVE IN THE EUROPEAN COMMUNITY/EUROPEAN UNION"

Symbol for "DO NOT USE IF PACKAGE IS DAMAGE" Symbol for "KEEP OUT OF THE REACH OF CHILDREN" Symbol for "MANUFACTURER"

Symbol for "TEMPERATURE LIMIT" Symbol for "KEEP AWAY FROM SUNLIGHT" Symbol for "MEDICAL DEVICE"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding. For feedback, complaints and reporting of unexpected events contact customer care.