

Hiflex Impression Paste

Zinc Oxide - Eugenol Based Impression Paste

PREVEST DenPro®
THE FUTURE OF DENTISTRY

Product Description : Hiflex Zinc Oxide - Eugenol impression paste is an irreversible rigid impression material used for recording impression of edentulous patients with the help of muco compressive technique. The mix material adheres well to impression tray & has adequate rigidity to prevent distortion of the impression borders. It has good dimensional stability, good detailed reproduction and easy to manipulate.

Composition:

Base Paste: Zinc Oxide, Vegetable Oil.

Catalyst Paste: Eugenol, Fillers and Accelerators

Intended Purpose/ Intended Use: It is intended to be used as an impression material for detailed reproduction of soft oral tissues.

Indications for Use:

- ◆ Impressions for completely edentulous cases.
- ◆ Impressions for partially edentulous distal extension.
- ◆ Relining of denture base.

Instructions for Use :

1. Instruct the patient to rinse the mouth thoroughly to remove excess saliva. Lightly lubricate the patient's lips and mouth corners with petroleum jelly.
2. Dispense equal length of base and catalyst pastes on a mixing pad. Use stainless steel spatula for mixing the pastes for 30-40 seconds until uniform consistency is obtained.
3. The mixture is transferred into previously made impression tray. Place the tray in patient's mouth. Begin muscle trimming in the usual manner approximately for 30 seconds after seating the impression tray. The paste sets hard in 3-5 minutes approximately. Verify complete set and remove the impression tray.
4. Make the patient rinse thoroughly with water to ensure no residues are left in the oral cavity. The dispensed material shall not be placed back in container and the dose once applied and used shall not be reused.

Warnings:

1. Avoid contact with the skin and the eyes. In case of contact flush with copious amount of water. If irritation persists, seek medical attention.
2. Do not swallow or take internally. If accidental swallowing occurs, drink copious amount of water. Seek medical attention.

Precautions:

1. The product is intended to be used by the qualified dental professionals only.
2. The material contains eugenol and may occasionally cause mucosal irritation to the patient. Therefore, do not leave residues of the impression material in the oral cavity.
3. Replace the cap tightly onto the respective tubes, that is, base paste cap on base tube and catalyst paste cap on catalyst tube immediately after use.

Contraindications: It contains eugenol and should not be used in patients with history of severe allergic reactions to eugenol and any other components of this product.

Storage Conditions : Store at temperature between 10°C(50°F) to 24°C(75°F). Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

Shelf Life : 3 years from the date of manufacturing.

Presentation :

Ref. : 70017

1 x 140g Base Paste, 1 x 70g Catalyst Paste.

Disposal Conditions : To be disposed off based on the prevailing rules of the country in which it is used.

Prevest DenPro Limited

Unit II, Export Promotion Industrial Park,
(EPIP), Bari Brahmara, Jammu-181133,
India. www.prestveddenpro.com
Customer Care No. : + 91 8899074151
Email : customercare@prevestdenpro.com
Made in India

E C R E P

Obelis s.a

Boulevard General Wabis 53
1030 Brussels, Belgium
Tel: + (32) 2 732 59 54, Fax: + (32) 2 732 60 03
E-mail : mail@obelis.net



Revision :	02
Date :	28-11-2025

[MD] Symbol for "MEDICAL DEVICE" [UDI] Symbol for "UNIQUE DEVICE IDENTIFIER"

[LOT] Symbol for "BATCH CODE" [USE-BY DATE] Symbol for "NON STERILE" [KEEP DRY]

[CAUTION] [CONSULT INSTRUCTION FOR USE] [REF] Symbol for "CATALOGUE NUMBER"

[EC/REP] Symbol for "AUTHORIZED REPRESENTATIVE IN THE EUROPEAN COMMUNITY/EUROPEAN UNION"

[KEEP OUT OF THE REACH OF CHILDREN] [MANUFACTURER] [DO NOT USE IF PACKAGE IS DAMAGE]

[TEMPERATURE LIMIT] [KEEP AWAY FROM SUNLIGHT] [DO NOT REUSE]

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding. For feedback, complaints and reporting of unexpected events contact customer care.