

Hiflex Impression Paste

Eugenol Free Impression Paste

PREVESTDenPro®
THE FUTURE OF DENTISTRY

Product Description : Hiflex Eugenol free impression paste is an irreversible rigid impression material used for recording impression of edentulous patients with the help of muco compressive technique. The mix material adheres well to impression tray & has adequate rigidity to prevent distortion of the impression borders. It does not contain eugenol and is therefore irritant free with better taste.

Composition:

Base Paste: Zinc Oxide, Vegetable Oil.

Catalyst Paste: Fillers and Accelerators

Intended Purpose/ Intended Use: It is intended to be used as an impression material for detailed reproduction of soft oral tissues in edentulous jaw.

Indications for Use:

- ◆ Impressions for completely edentulous cases.
- ◆ Impressions for partially edentulous distal extension.
- ◆ Relining of denture base.

Instructions for Use :

1. Instruct the patient to rinse the mouth thoroughly to remove excess saliva. Lightly lubricate the patient's lips and mouth corners with petroleum jelly.
2. Dispense equal length of base and catalyst pastes on a mixing pad. Use stainless steel spatula for mixing the pastes for 30-40 seconds until uniform consistency is obtained.
3. The mixture is transferred into previously made impression tray. Place the tray in patient's mouth. Begin muscle trimming in the usual manner approximately for 30 seconds after seating the impression tray. The paste sets hard in 3-4 minutes approximately. Verify complete set and remove the impression tray.
4. Make the patient rinse thoroughly with water to ensure no residues are left in the oral cavity. The dispensed material shall not be placed back in container and the dose once applied and used shall not be reused.

Warnings:

1. Avoid contact with the skin and the eyes. In case of contact flush with copious amount of water. If irritation persists, seek medical attention.
2. Do not swallow or take internally. If accidental swallowing occurs, drink copious amount of water. Seek medical attention.

Precautions:

1. The product is intended to be used by the qualified dental professionals only.
2. Do not leave residues of the impression material in the oral cavity.
3. Replace the cap tightly onto the respective tubes, that is, base paste cap on base tube and catalyst paste cap on catalyst tube immediately after use.

Contraindications: Do not use in patients with history of severe allergic reactions to any of the components of this product.

Storage Conditions : Store at temperature between 10°C(50°F) to 24°C(75°F). Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

Shelf Life : 3 years from the date of manufacturing.

Presentation :

Ref. : 70018

1 x 140g Base Paste, 1 x 70g Catalyst Paste.

Disposal Conditions : To be disposed off based on the prevailing rules of the country in which it is used.

Prevest DenPro Limited

Unit II, Export Promotion Industrial Park,
(EPIP), Bari Brahmna, Jammu-181133,
India. www.prestveddenpro.com

Customer Care No. : + 91 8899074151

Email : customer@prevestdenpro.com

Made in India

E C R E P

Obelis s.a

Boulevard General Wahis 53

1030 Brussels, Belgium

Tel: + (32) 2 732 59 54, Fax: + (32) 2 732 60 03

E-mail: mail@obelis.net



Revision :	01
Date :	26-04-2025

[MD] Symbol for "MEDICAL DEVICE" [UDI] Symbol for "UNIQUE DEVICE IDENTIFIER"

[LOT] Symbol for "BATCH CODE" [USE-BY DATE] Symbol for "USE-BY DATE" [NON STERILE] Symbol for "NON STERILE" [KEEP DRY] Symbol for "KEEP DRY"

[CAUTION] Symbol for "CAUTION" [CONSULT INSTRUCTION FOR USE] Symbol for "CONSULT INSTRUCTION FOR USE" [REF] Symbol for "CATALOGUE NUMBER"

[EC REP] Symbol for "AUTHORIZED REPRESENTATIVE IN THE EUROPEAN COMMUNITY/EUROPEAN UNION"

[KEEP OUT OF THE REACH OF CHILDREN] Symbol for "KEEP OUT OF THE REACH OF CHILDREN" [MANUFACTURER] Symbol for "MANUFACTURER" [DO NOT USE IF PACKAGE IS DAMAGE] Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"

[TEMPERATURE LIMIT] Symbol for "TEMPERATURE LIMIT" [KEEP AWAY FROM SUNLIGHT] Symbol for "KEEP AWAY FROM SUNLIGHT" [DO NOT REUSE] Symbol for "DO NOT REUSE"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding. For feedback, complaints and reporting of unexpected events contact customer care.