

Silane-X

Silane Coupling Agent

PREVESTDenPro®
THE FUTURE OF DENTISTRY

Product Description : Silane-X is a single component material which is used to enhance bond strength between composites/cements and porcelain. For best results the etched porcelain surface should be completely dry and free of residual moisture before applying Silane-X.

Composition : Organo functional Silane and Solvents.

Intended Purpose / Intended Use : It is used to be applied to dental prosthesis prior to insertion into a tooth structure to promote bonding to a prosthesis components.

Indications for Use : Used for porcelain repairs and bonding etched porcelain restorations.

Instructions for Use (Cementation of ceramic restorations) :

1. Clean the area to be cemented.
2. Carefully etch the porcelain surface, rinse and dry as instructed.
3. Apply Silane-X Bond Enhancer with a mini brush tip to treated porcelain surface.
4. Allow to dry for one minute. Apply gently moisture free air stream to accelerate drying.
5. Apply bonding resin and proceed with restoration using composites or resin cements.

Instructions for Use (Intraoral repair) :

1. Isolate the area to be repaired with a rubber dam.
2. Gingival Barrier may be used to protect areas where etching is not desired.
3. Carefully etch the porcelain surface, rinse and dry as instructed.
4. Apply Silane-X Bond Enhancer with a mini brush tip to treated porcelain surface. Allow to dry for one minute. Apply gently moisture free air stream to accelerate drying.
5. Apply bonding resin and proceed with restoration using composites or resin cements.
6. The dispensed material shall not be placed back in the container and the dose once applied or used shall not be reused.

Warnings : Silane-X contains flammable solvents. Keep away from ignition source.

Precautions :

1. One must remove the tip & re-cap the syringe after each use. Always verify material flow prior to applying intraorally.
2. Clinician and patient must wear protective eyewear.
3. Avoid contact with skin, this may cause skin sensitization in susceptible persons.
4. For professional use only.

Contraindications : Do not use in patients, If allergic to any of its components.

Storage Conditions : Store at temperature between 10°C(50°F) to 24°C(75°F). Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool and dry place.

Shelf Life : 2 years from the date of manufacturing.

Presentation :

Ref: 10014

Intro Pack : 1 x 1.2ml Syringes + 5 Applicator Tips.

Ref: 10015

Economy Pack : 1 x 10ml Plastic Bottle.

Disposal conditions : To be disposed off based on the prevailing rules of the country in which it is used.

Prevest DenPro Limited
Unit II, Export Promotion Industrial Park
(EPIP), Bari Brahmam, Jammu-181133,
India. www.prevestdenpro.com
Customer Care No. : + 91 8899074151
Email : customercare@prevestdenpro.com
Made in India

EC REP
Obelis s.a
Boulevard General Wahis 53
1030 Brussels, Belgium
Tel: +32 2 732 59 54, Fax: +32 2 732 60 03
E-mail : mail@obelis.net

CE
0123

Revision :	08
Date :	23-03-2026

UDI Symbol for "UNIQUE DEVICE IDENTIFIER"

LOT Symbol for "BATCH CODE"  Symbol for "USE-BY DATE"  Symbol for "NON STERILE"  Symbol for "KEEP DRY"

 Symbol for "CAUTION"  Symbol for "CONSULT INSTRUCTION FOR USE"  Symbol for "CATALOGUE NUMBER"

 Symbol for "AUTHORIZED REPRESENTATIVE IN THE EUROPEAN COMMUNITY/EUROPEAN UNION"

 Symbol for "DO NOT USE IF PACKAGE IS DAMAGED"  Symbol for "KEEP OUT OF THE REACH OF CHILDREN"  Symbol for "MANUFACTURER"

 Symbol for "TEMPERATURE LIMIT"  Symbol for "KEEP AWAY FROM SUNLIGHT"  Symbol for "MEDICAL DEVICE"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding. For Feedback, complaints and reporting of unexpected events contact customer care.