

ZINCONOL XS

POLYMER REINFORCED ZOE CEMENT

PREVESTDenPro®
THE FUTURE OF DENTISTRY

Product Description :

Zinconol Extra Strong (Zinconol XS) is a polymer reinforced zinc oxide eugenol cement for temporary restoration. The excellent physical properties resulting from the unique polymer reinforcing agents include extra strong intermediate restoration and tensile strength, excellent abrasion resistance and low solubility.

Composition :

Powder contains Zinc Oxide, Fillers and Accelerators and Liquid contains Eugenol and EBA

Intended Purpose/Intended Use : It is intended to be used as a temporary filling material

Indications for Use :

Intermediate restorative material (IRM) for class I & II restorations. Temporary restoration in emergency treatments and to seal the coronal portion of the teeth under endodontic treatment. As a base under amalgam and non resin restorations.

Instructions for Use :

Shake the bottle of powder and liquid well before use to ensure that the density is homogeneous. Powder and liquid should be mixed in proper ratio. Powder liquid ratio is 1 : 2 (2 scoop powder and 2-3 drops of liquid.) Mixing should be completed in 45- 60 seconds to achieve a putty consistency. Insert mix material into the cavity and allow 4-6 minutes for proper hardening. The dispensed material shall not be placed back in the container and the dose once applied or used shall not be reused.

Warnings :

Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take internally. Contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs, immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs, discontinue the use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician.

Precautions : Zinconol XS is a Eugenol based composition and therefore it should not be used as a liner under resin based filling materials. Do not invert the bottle upside down while opening to avoid excess dispensing of the powder and avoid excessive force while dispensing the liquid from the dropper.

Contraindications : It is contraindicated for use in patients who have a history of severe allergic reaction to any of the components.

Storage Conditions :

Store at temperature between 10°C (50°F) and 24°C (75°F). Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

Shelf Life :

3 years from the date of manufacturing.

Presentation :

Ref : 30022

1 x 38g Powder, 1 x 14ml Liquid, 1 x Spoon, 1 x Mixing Pad and 1 x Dispensing Dropper .

Disposal conditions : To be disposed off based on the prevailing rules of the country in which it is used.

Prevest DenPro Limited

Unit II, Export Promotion Industrial Park (EPIP),

Bari Brahmana, Jammu-181133, India.

www.prevestdenpro.com

Customer Care No. : + 91 8899074151

Email : customer@prevestdenpro.com

Made in India

Revision : 01

Date : 23-09-2025

UDI Symbol for "UNIQUE DEVICE IDENTIFIER"

LOT Symbol for "BATCH CODE" USE-BY DATE Symbol for "USE-BY DATE" NON STERILE Symbol for "NON STERILE" KEEP DRY Symbol for "KEEP DRY"

CAUTION Symbol for "CAUTION" CONSULT INSTRUCTION FOR USE Symbol for "CONSULT INSTRUCTION FOR USE" REF Symbol for "CATALOGUE NUMBER"

EC REP Symbol for "AUTHORIZED REPRESENTATIVE IN THE EUROPEAN COMMUNITY/EUROPEAN UNION"

DO NOT USE IF PACKAGE IS DAMAGED Symbol for "DO NOT USE IF PACKAGE IS DAMAGED" KEEP OUT OF THE REACH OF CHILDREN Symbol for "KEEP OUT OF THE REACH OF CHILDREN" MANUFACTURER Symbol for "MANUFACTURER"

TEMPERATURE LIMIT Symbol for "TEMPERATURE LIMIT" KEEP AWAY FROM SUNLIGHT Symbol for "KEEP AWAY FROM SUNLIGHT" MD Symbol for "MEDICAL DEVICE"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding. For feedback, complaints and reporting of unexpected events contact customer care.